

Buscot Babevac Cot ^{CE}



(Part of the “SKI” range of products)

Designed for the safe evacuation of premature and new born babies from maternity wards and special care baby units in the event of fire or emergency.

Features:

- Can carry up to two babies depending on size and condition.
- Enables carers to be ‘hands free’ to administer needed care to the infant while evacuating.
- Secure storage for one size CD oxygen cylinder on the underside of the cot, to allow manual ventilation of baby if required (Cylinder not included).
- Flexible strong contraction allows for easy handling and adequate support and protection of babies.
- Adjustable shoulder straps.
- Wall mountable for easy, instant access.
- Easy wipe clean after use



Using the Buscot Babevac Cot:

- Remove the Buscot from its storage bag and unfold.
- Ensure that the cot is on a flat surface
- Swaddle up the baby/babies for warmth and place inside the cot.
- Place the straps over the shoulder - creating a single cross over the chest area and fasten using the plastic safety buckles.
- If manual ventilation is needed, a second carer would need to secure the cylinder to the underside of the Buscot.
- Move the baby/babies within the Buscot, to a safe area.

Specifications:

Packed: 62cm x 36cm x 9cm
Opened: 58cm x 30cm x 28cm
Padding: 5mm
Weight: 2kg
Material: Vinyl
Cylinder capacity: Size CD (not included)

Product code: BUS001



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553

