


The ResQmat enables designated personnel to rapidly and safely evacuate people with mobility problems in an emergency. The built in padding and extra pillow under the head means that the ResQmat is safe and comfortable for people when going down stairs.

Features & Benefits:

- 3cm thick padding
- 10 cm thick pillow.
- 3 wide Velcro and webbing cross straps.
- Size adjustable to suit any height from child to adult.
- Pulling handles at head and foot end.
- Wall mountable, wipe clean storage bag.
- No metal parts – MRI and X-Ray safe

Using the ResQmat:

- In an Emergency the ResQmat is quickly removed from its wall mountable storage bag.
- The patient can then be transferred on to the ResQmat, making sure the pillow is at the head end and feet should be within the V section of the red strap.
- Firmly wrap the side wings of the ResQmat over the patient and secure with the Velcro straps, starting at the foot end.
- Pull the red strap through the plastic buckle and fasten for a tight fit.
- Using the end pulling handles the individual can be moved to a safe area. When going down stairs it is important to pull the individual foot end first.

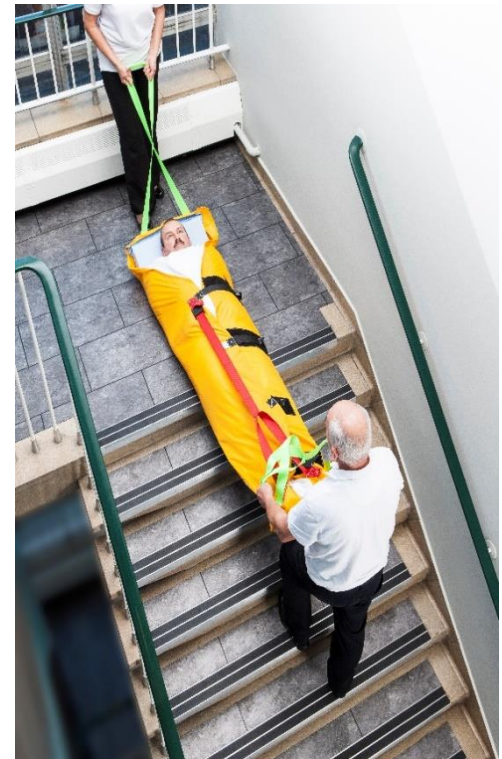

Specifications:

Packed: 51cm x 51cm x 22cm

Unrolled: 225cm x 48cm with two side wings 41cm

Padding: 3cm thick

Weight: 4kg

Base: Polypropylene

Top: Yellow nylon

Safety tested to: 160kg

Safe working load: number of handlers needed should be based on a risk assessment



Product code: RES001

Replacement Covers: RES002

Onsite product demonstrations and training packages are available please contact us for more information.



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553

