

SMOKERS PROTECTIVE COVER ^{CE}



Heavy duty flame retardant apron designed to protect smokers from burns due to fallen hot ashes and dropped cigarettes. Ideal for use by individuals suffering with Parkinsons or Huntingdons disease.

Features & Benefits:

- Designed to cover user's chest, knees and chair arms, offering a large area of protection.
- Securely attached by press studs, but quickly removed if necessary to smother fire.
- Side panels to prevent cigarettes etc. from slipping down between the chair and user.
- Does not shrink, melt or drip in open flame.
- Fabric is designed to char but will NOT ignite.
- Made from a Kevlar and Nomex Inherently flame resistant material - inherently flame resistant Material has protection built into the fiber itself and this protection can never be worn away or washed out.

(TITAN fabrics comply with international standards, including Structural Firefighting – EN469 – NFPA 1971 – AS/NZS 4967 Technical Rescue and EMS – NFPA 1951 – NFPA 1999 Police – BS7971-10.)

Using the Smokers Cover:

- Lay Smokers Cover on intended user.
- Fasten the neck strap and adjust if required.
- Fasten top elastic around the backrest of the chair, insuring it passes under the user's arms.
- Fasten the press studs down the left and right front sides to form a box shape.
- Fasten the bottom elastic around the back chair legs and fasten to the cover.
- If the chair has no legs then fasten the elastic to the cover and leave loose.

Laundering Instructions:

- Washable at 60° C.
- Tumble Dry on low heat until totally dry
- Can be ironed if required with or without steam.
- Do not bleach

Product Code: SMOK01



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553

